



Mona Complex

SHR-950S Mini Hair Removal System

IPL + E-Light+SHR

Introduction Of SHR-950S



① SR handpiece



② HR handpiece



③ Handle plug seat

Complex System of SHR-950S



IPL Mode




E-Light Mode



SHR Mode

Application of SHR-950S

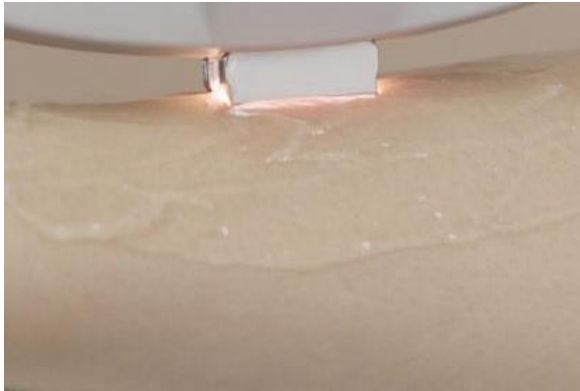

- **Everlasting Hair Removal**

The unwanted hair under arms, legs, lips, or other body parts.

- **Skin Rejuvenation**

Shrink pores, tighten skin, improve skin elasticity and glossiness.

- **Pigmentation Removal**

Removal various kinds of pigment such as freckle, chloasma, aged spots, sunburn, etc.

- **Acne Clearance**

Specification of SHR-950S

Wave Length	HR: 650-950nm Optional:(SR: 560-950nm VR 420-950nm PR: 530-950nm AR:500-950nm)
Fluence	IPL&E-light: 10-50J/cm ² SHR: 1-10J/cm ²
Frequency	IPL&E-light: ≤0.5HZ SHR: 1-8HZ SSR:1-3HZ
RF power	1-30W
Input Power	3000W
Power Source	Pure Sapphire
Lamp	Imported lamp
Spot Size	10*40mm for(SR,VR,AR,PR) 15*50mm for HR
Contact cooling temperature	Max -10°C
Cooling System	Built in water cooling +semi conductor cooling +air cooling
LCD Screen	10.4 inch true color touch screen control
Weight	30kg
machine size	633 (L) * 396 (H) * 460 (W)
package size	693 (L) * 460 (H) * 550 (W)


Work Theory of SHR-950S

- The SHR950S combines **IPL, E-light and SHR** technology in one system to satisfy different treatment requirement.
- SHR mode gently heats the hair follicle and the clients feeling is only warmth and a tingling sensation, some clients comparing it to a warm massage.
- SHR mode also utilizes In-Motion technology, where the hand piece is always in motion over the skin.



Advantage and System Feature

- SHR-950S Complex is a complete system for IPL Hair removal treatment. It includes the traditional IPL and elight working mode to treat your skin with high amounts of Joules (energy), and also includes SHR working mode to shot multiple shots but at low amounts of Joules.
- SHR Mode combined with “In Motion” represents a breakthrough in permanent hair removal with light technology. The treatment is more pleasant than with the conventional systems and the skin is better protected.
- The system is suitable for all skin types, even tanners; Unlike traditional Laser and IPL, SHR can be had all year round without having to hide from the sun.
- The system utilize imported flash lamp and has 300, 000 shots life span. The treatment head is made of pure sapphire crystal.

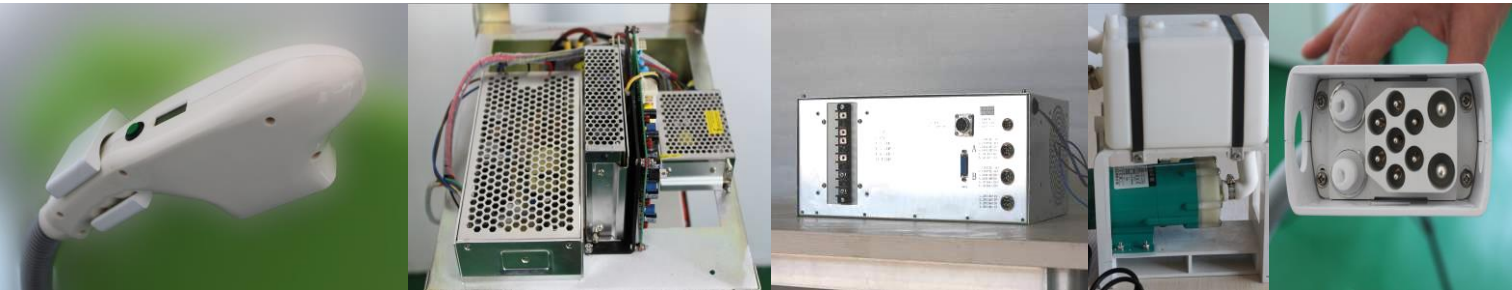


- The wavelength of 650-950nm for Hair removal and 560-950nm for skin rejuvenation are utilized. The special made crystal filters the violet light and infrared light, which can eradicate the absorption of water from target skin.
- SHR-950s is equipped with water temperature sensor, water level sensor and water flow sensor to ensure the safe operation. It can automatically stop working when the machine has potential problem.
- SHR 950s is a Multi pulse & single pulse IPL machine. Only 4-5 times is needed to remove unwanted hair and the client won't feel painful.

Take part in our design, make your own colour !



Difference	Traditional IPL	SHR 950S	Advantage
Wavelength	650-1200nm(HR) 560-1200nm (SR)	650-950nm(HR) 560-950nm(SR)	SHR-950S filter violet light and infrared light, which could radiate and absorb water from target skin.
Auto identified handpiece	Unavailable	Available	Identified hand piece, HR &SR. The system can enter into correct operation menu directly when you insert the hand piece. Identified function makes the operation simple and treatment safety.
Frequency	0.3-1Hz	1-8Hz	Which can speed up the operation and reduce the treatment time.
Work mode	Stationary mode	Stationary mode & Moving mode	1/8 time is needed for treatment on back, leg and arm comparing with traditional IPL treatment.
Water circuit monitor	Unavailable	Available	Water temperature sensor, water flow sensor and water level sensor make sure the machine to work safely without worry about hand piece burned or burning patients.
Stain resistant water circle	Unavailable	Available	Stain resistant water circuit system make sure all the components no rust happens even long time system running. This makes the crystal and filter cleaning and use longer time.
Water lock connector	Unavailable	Available	Treatment head connector with water-lock for safe and easy changing, even with only single treatment head use is allowed.

BEFORE & AFTER TREATMENT

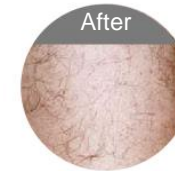

Welcome To Visit



www.laser-ipl-rf.com, www.sanhebeauty.com

Tel: +86-10-53006037

Beijing Sanhe Beauty S & T Co.,Ltd